

# AccuSign™

# DOA Series

## New One-Step Urine Tests For Drugs of Abuse




### Test Procedure

- ▶ Collect urine in clean cup.
- ▶ Open the cassette and dip in the urine cup upto the mark.
- ▶ Read result in 5 minutes (within 10 minutes)

### Reading the Result (REVERSE PRINCIPLE)

Negative : A specific band for each drug tested, plus a Control Band (C)

Positive : No specific band for each drug tested. (Control band only)

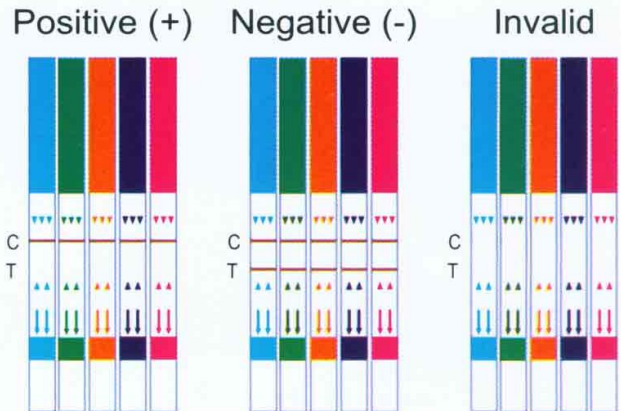

### Ordering info....

Cat No.	Product Name	Pack Size
DO1ASS30	AccuSign AMP Strips	30 Tests
DO1ASS50	AccuSign AMP Strips	50 Tests
DO1ASC30	AccuSign AMP Cards	30 Tests
DO2ASS30	AccuSign BAR Strips	30 Tests
DO2ASS50	AccuSign BAR Strips	50 Tests
DO2ASC30	AccuSign BAR Cards	30 Tests
DO3ASS30	AccuSign BZO Strips	30 Tests
DO3ASS50	AccuSign BZO Strips	50 Tests
DO3ASC30	AccuSign BZO Cards	30 Tests
DO4ASS30	AccuSign COC Strips	30 Tests
DO4ASS50	AccuSign COC Strips	50 Tests
DO4ASC30	AccuSign COC Cards	30 Tests
DO5ASS30	AccuSign OPI Strips	30 Tests
DO5ASS50	AccuSign OPI Strips	50 Tests
DO5ASC30	AccuSign OPI Cards	30 Tests

Cat No.	Product Name	Pack Size
DO6ASS30	AccuSign PCP Strips	30 Tests
DO6ASS50	AccuSign PCP Strips	50 Tests
DO6ASC30	AccuSign PCP Cards	30 Tests
DO7ASS30	AccuSign THC Strips	30 Tests
DO7ASS50	AccuSign THC Strips	50 Tests
DO7ASC30	AccuSign THC Cards	30 Tests
DO8ASC05	AccuSign DOA 5 Panel (COC / OPI / THC / AMP / MET)	30 Panel Tests
DO9ASS30	AccuSign MET Strips	30 Tests
DO9ASS50	AccuSign MET Strips	50 Tests
DO9ASC30	AccuSign MET Cards	30 Tests
D10ASC05	AccuSign DOA 5 Panel (COC / OPI / THC / AMP / BAR)	30 Panel Tests



# AccuSign™

# DOA Series

## New One-Step Urine Tests For Drugs of Abuse

Amphetamines

Barbiturates

Benzodiazepines

Cocaine

Methadone

Methamphetamines

Opiates

Phencyclidine

THC (Cannabis/Marijuana)


Tricyclic Antidepressants

Drug Panels  
(DOA2, DOA3, DOA4 & DOA5)

### Test Procedure


#### Card

1. Open the AccuSign DOA pouch & dispense 2 drops (25 µL each) of the urine sample into Sample well (S) of the device.
2. Read the result in 3-10 minutes.

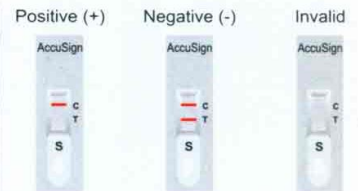

#### Strip

1. Open the AccuSign DOA pouch and dip the strip in the urine at RT upto the arrow mark only.
2. Take the strip out of the urine once the sample is seen moving up on the membrane.
3. Read the result in 2-5 minutes (within 10 minutes).

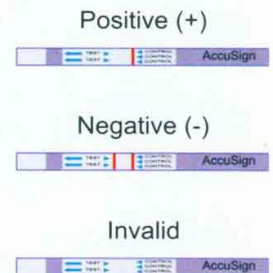

### SALIENT FEATURES

- Simple : One Step
- Rapid : Results within 10 minutes
- No need to add accurate volume
- Highly Sensitive
- Easy to read colour
- Built-In Test Control
- Complete Walkaway Capability
- Room Temperature Storage
- Eliminates Timing
- Simultaneous Testing Capability of Multi-drugs with individual tests or with the DOA Panel

Committed to improve Quality of Life



ISO 9001:2008 CERTIFIED ORGANISATION

- \*WHO-GMP Certified Manufacturing Facility
- \* ISO 9001-2008 Certified Company
- \* ISO 13485:2003 Certified Company



### Premier Medical Corporation Limited

32-35, Shree Ganesh Indl. Estate, Kachigam, Nani Daman, Daman - 396 215, INDIA  
Tel.: +91 260 3298483, 3266800 • Telefax : +91 260 2242411  
email : info@premiermedcorp.com • www.premiermedcorp.com