




Premier Medical Corporation Limited

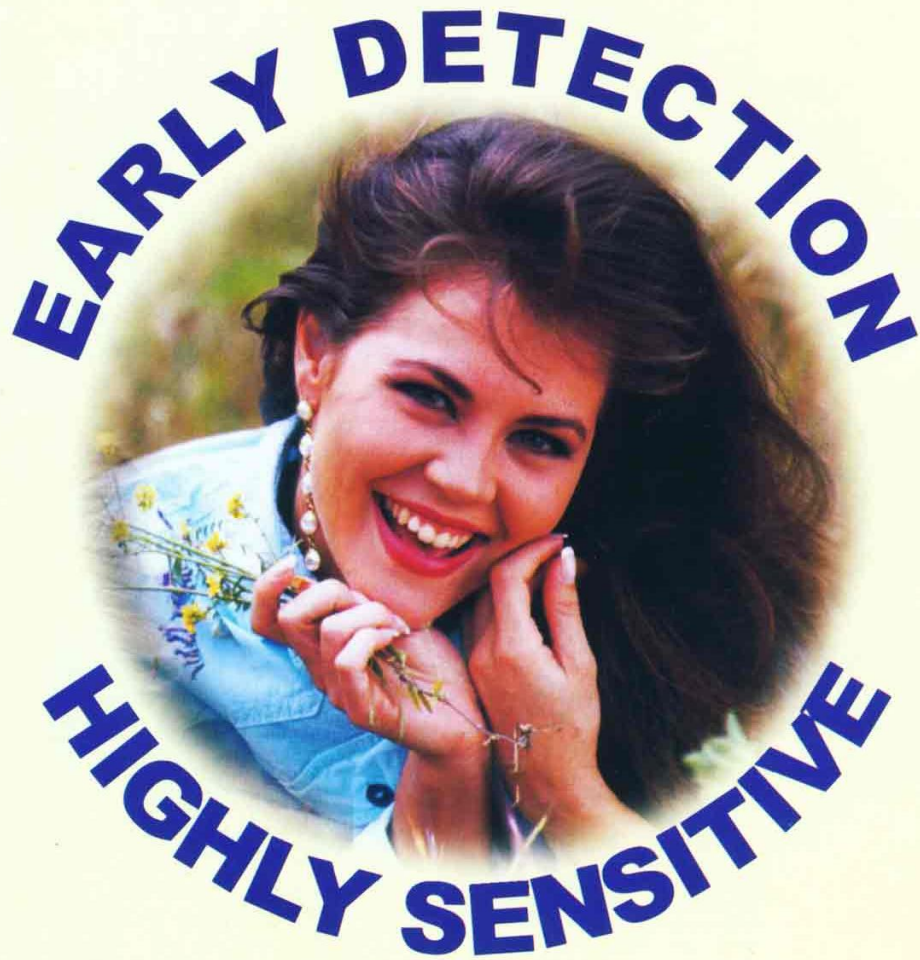

AQCheck hCG

Fast & Accurate

The 2 Minutes Pregnancy Test



One-Step Pregnancy Card Test

AQCheck hCG

Read in 2-10 minutes

Add 2 drops of urine sample (35 µL each)

1 Line = Negative (-) (Not Pregnant) 2 Lines = Positive (+) (Pregnant) Invalid

✓ 100% Sensitivity

✓ 100% Specificity

✓ 100% Accurate

One-Step Pregnancy Strip Test

AQCheck hCG

1. Add 1 mL of urine sample to the test tube
2. Dip the AQCheck hCG in the urine upto the arrow mark.
3. Take the strip out of the urine once the sample is seen moving up on the membrane. It takes 25 to 30 seconds. Place the strip on a dry flat surface.
4. Read the result in 2-5 minutes (within 10 minutes)

Positive Two Lines Negative One Line Invalid



Premier Medical Corporation Limited

32-35, Shree Ganesh Indl. Estate, Kachigam, Nani Daman, Daman - 396 215, INDIA

Tel.: +91 260 3298483, 3266800 • Telefax : +91 260 2242411

email : info@premiermedcorp.com • www.premiermedcorp.com



ISO 9001:2008 + ISO 13485:2003



1434



ISO 9001:2008 CERTIFIED ORGANISATION

*WHO-GMP Certified Manufacturing Facility

* ISO 9001-2008 Certified Company

* ISO 13485:2003 Certified Company